

HSDW-1 - FAQ's

Wall-Mount Sanitiser Station & Automatic No Touch Dispenser

1. The instructions are confusing

A: Updated assembly instructions have been added to the web page:
www.visionchart.com.au/sanitiser-station.html

2. I can't find the Dispenser unit bracket (a.1)

A: (a.1) is pre-attached to the back of (a), found in the Dispenser unit box.

3. The lid wont lock down?

A: The lid will not lock into position if the Dispenser unit (a) is not correctly secured to the bracket (a.1). Ensure that the Dispenser unit is secured correctly.

4. Unit won't turn on, I have put the batteries in

A: Make sure that on the bottom of the Dispenser unit the 2 dots have been lined up, and the battery compartment is securely locked in correctly. Shown in updated instructions.

5. Why is the unit beeping?

A: Beeping will occur when one battery has been placed into the unit the wrong way around. Ensure that batteries are the right way around.

6. Sanitiser and unit: What and how to use?

A: The unit only works with GEL sanitiser. GEL sanitiser is thicker than other sanitisers. Watery, runny non-gel sanitiser won't work in the unit, and will cause a spray and dribble effect. We suggest to use Clean Plus Instant GEL Hand Sanitiser.

Once the stand is assembled – turn the top key (b) which will open up the storage tank for the GEL, pour in the GEL, and to relock turn the key again. Do not throw away the Key (b).

On the bottom of the Dispenser unit there are 2 Buttons: One is the on/ off button and the other is for the flow control. There are 3 settings available. Turn off the unit to start with, turn it back on and then press the button once/ twice/ three times depending on the amount you would like to release as per settings.

7. The unit won't work at all

A: The checklist is:

1) Check batteries are installed correctly 2) Check Unit is turned on 3) Check that nothing is impeding the sensor

If the unit attempts to dispense but doesn't – check that you are using a GEL sanitiser

If the unit does not attempt to dispense – check the power.

If you have any further questions, please contact us.

